Inhaltsverzeichnis / Contents:

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Please fix the skai-padding with velcro as follows.

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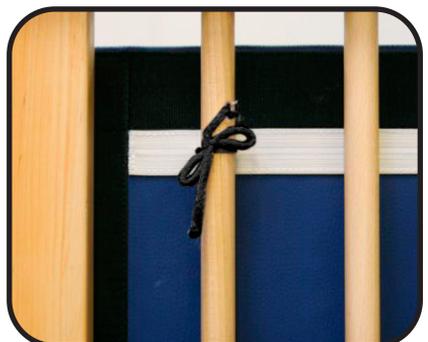
Please fix the skai-padding with the cords as follows.

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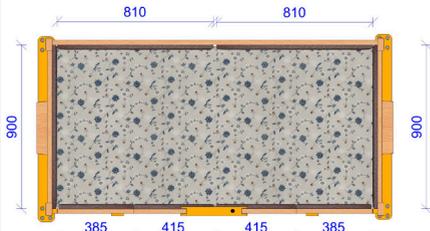
Please fix the skai-padding with velcro and sticky back as follows.

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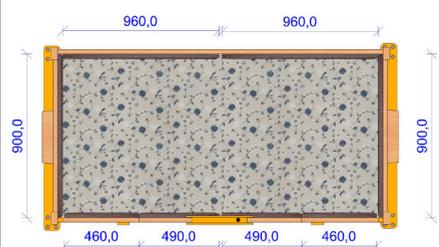
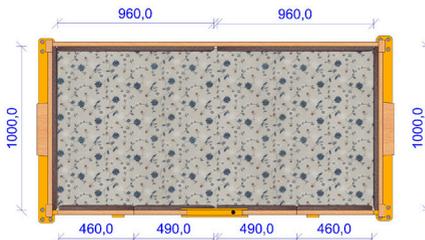
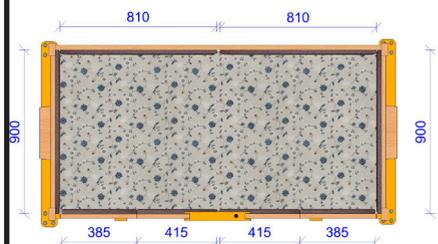
Operating instruction for our skai-padding

Articel-No.: 99.051

(lying area 90x170 cm)


Contains:

- 2 padding 810 x 500 mm (BxH)
- 2 padding 900 x 500 mm (BxH)
- 2 padding 385 x 500 mm (BxH)
- 2 padding 415 x 500 mm (BxH)

B = width

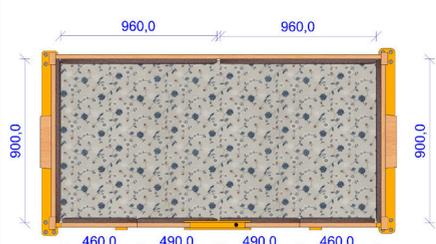(lying area 90x200 cm)


Contains:

- 2 padding 960 x 500 mm (BxH)
- 2 padding 900 x 500 mm (BxH)
- 2 padding 460 x 500 mm (BxH)
- 2 padding 490 x 500 mm (BxH)

H = Height

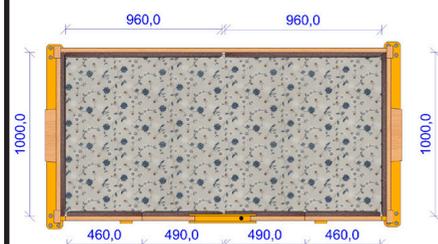(lying area 100x200 cm)

Contains:

- 2 padding 960 x 500 mm (BxH)
- 2 padding 1000 x 500 mm (BxH)
- 2 padding 460 x 500 mm (BxH)
- 2 padding 490 x 500 mm (BxH)

Please take the arrangement of skai-padding as is discribed above.

You can fix the skai-padding individually because there are 3 diffent versions.

NOTE:

Before the doors of the children's care bed can be opened, you must remove the skai-padding in this area. Also, the electric height adjustment can be used only if the skai-padding have been previously removed.

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Cleaning and care:

Artificial leather has softener and malleable making additives that provide it with a supple property. A professional and competent cleaning is essential to maintain the colours and to prevent material hardening.

For lightly soiled material we suggest cleaning with a soft cloth and soapsuds. Single spots (for example ballpen ink) have to be treated immediately with alcoholic detergent with at least 50 % alcohol.

Never use aggressive chemical detergents or abrasives!

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SAVI GmbH Reha Solutions
Lise-Meitner-Str. 10
D-48691 Vreden

www.savi.de

info@savi.de

FACEBOOK: SAVI.GmbH

Fon: +49 (0) 25 64 - 93 09-70

Fax: +49 (0) 25 64 - 93 09-60